

Infectrol[®] GF

Manufactured in GMP unit

Made in India

Product of
Indigenous
Research
& Inputs

Other Range

Aldefree

Ultrabasil[®] Plus

Enzycide[®]

Disdril

Remorust

Microbat[®]



UPS Hygienes Pvt. Ltd.

(AN ISO 9001-2015 Certified Company)

Customer Care No.: 022-65214567 / 9821338361

E-mail: info@upshygiene.com

Your Partner In Infection Control

Combination is 10 times more lethal than either chemical alone¹

Good material compatibility, presence of antirust agent.

No wastage, Prepare as & when required.

Contains surfactant cleaners, helps for precleaning.

Gluteraldehyde at slightly acidic pH is more stable, more sporicidal & less damaging. Acid stabilized solution not only polymerises slowly but also kills dried spores within 20 minutes².

Very active bactericidal, fungicidal, tuberculocidal, virucidal & sporicidal.

High microbiocidal activity against spores. Effective antimycobacterial. Effective against B. subtilis/B. pumilis. Heat resistant spores like Clostridium tetani are killed.

Formaldehyde - Acts by alkylation with amino & sulfhydryl group of proteins & nucleic acids.

Composition

Gluteraldehyde 7% w/w

Formaldehyde 8% w/w

Packing

500ml

20 bott./carton

Both are **HIGH LEVEL DISINFECTANTS.**

Neutral emulsifying agent stabilizes & increases the activity.

Suitable for emergency disinfection.

Ideal for metal, glass, steel, porcelain, plastic, ceramic, rubber etc.

For dental instruments, endoscopes, forceps.

Built in corrosion inhibitor protects instruments from corrosion.

Make a 1 in 10 dilution with potable water & use. The equipment should be free from organic matter like blood, serum before dipping in the diluted INFECTROL solution.

Dip instruments for specified contact time & then take out. Rinse instruments thoroughly. Replace the solution after 14 days.

Disinfection - contact time 15-20 minutes; Sterilisation - contact time 6-8 hrs.

Wear gloves to avoid skin contact. Protect eyes from splashes, limit exposure time & use only in well ventilated area.

Ref. : 1. Goodman & Gilman page 963

2. Martindale's extra pharmacopoeia 13 page 796

Website : www.upshygiene.co.in

Helpline : 022-65214567 / 9821338361

